Generation Knöchelbandage

Ankle Support | Chevillère | Vendaje para el tobillo | Cavigliera

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Ankle supports relieve injured tissue structures (joints, ligaments, capsules and cartilage). The compression hereby supports the holding function of the ligaments and is thus relieving the joint. The pad provides a massage effect which reduces swelling tendencies or which helps to an earlier restraint of swelling.

Instructions for use:

Before putting on, open the Velcro fasteners.

Pull the BORT Generation Ankle Support over the foot.

Place your foot on the ground and close first the upper and then the lower Velcro fastener.

Indications:

Chronic post-traumatic or post-operative soft tissue irritations in the ankle area, distortion, ligament weakness, after removal of plaster cast, ligament injuries, arthrosis.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

Composition:

58 % polyamide, 40 % elastodies / latex, 2 % spandex

Important Note:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and may trigger allergic reactions.

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BORT GmbH | Postfach 1330 | D-71367 Weinstadt | www.bort.com