# Generation Rückenorthese

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
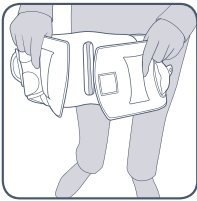
Braces in the lumbar spine region support damaged tissue structures (joints, capsules, ligaments and cartilages). Here, the circumferential frame supports the retaining function of the ligaments, thus relieving the lumbar spine. Through the combination of circumferential compression and stabilisation using semi-rigid brace half-shells, the region affected is relieved and supported. The BORT Generatio Back Brace was developed for patients who require a back brace which is easy to put on, is comfortable to wear and has a wide area of application.

### Instructions for use:

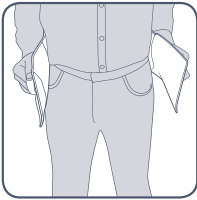
When you receive your BORT Generation Back Brace, the front fastener is closed. Open it fully.


Place the back brace around the torso so that the upper parts of the plastic elements are positioned above the iliac crest.

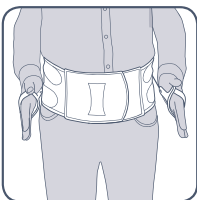Close the BORT Generation Back Brace applying very little force, using the attached hand straps.

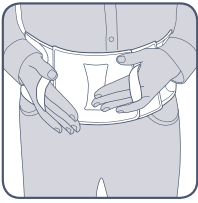In order to close the BORT Generation Back Brace fully, slip into the hand straps and pull the cable controls forwards evenly, applying very little force.

Following this, attach the hand straps to the velour using the micro Velcro provided.

Observe here that the patient still feels good and is not strangulated. Within the course of this fitting process, the BORT Generation Back Brace is fully fitted to the anatomy and interlocks.

If the lower part of the brace ends with the hip-bend when sitting, the brace is correctly positioned.

To take off the brace, detach the Velcro on the cable control hand straps and then open the front fastener.

### **Indications:**

Lumbosciatica, intervertebral disc protrusions / prolapse, radicular irritation syndrome, degenerative lumbar change, condition following intervertebral disc surgery, stable fractures, spondylosis, spondylolisthesis and spinal canal stenosis.

### **Contraindications:**

Lymphatic drainage disorders including unclear soft tissue swelling, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body.

### **Composition:**

84 % polyamide, 16 % spandex

### **Important note:**

No reutilisation – this medical device is meant for the care of one patient only. Wear splint only when under strain, not during rest or sleep. Avoid hindering blood circulation. In case of numbness loosen splint and remove it if necessary. If complaints continue, please contact the doctor.

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