Generation Rückenbandage

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Back supports support damaged tissue structures (bones, ligaments and cartilage). At the same time, the compression effect supports the retaining function of the local ligament and muscle apparatus, thus relieving the spine. The specially developed nodule pad supports the intercellular metabolism with its intermittent massage effect via increased circulation.

The Generation Back Support was developed for patients who require a support which is easy to put on, is very comfortable to wear and has a wide area of application.

Instructions for use:

When you receive your BORT Generation Back Support, the front fastener is closed. Open it fully.

The pad which is split into two in the back area must be positioned with the wider part of the pad in the direction of the head. Place both pads respectively on the outer edge of the support cover.

Place the back support around the abdomen and close the BORT Generation Back Support applying very little force, using the attached hand straps.

In order to close the BORT Generation Back Support fully, slip it into the hand straps and pull the pulleys forwards evenly, applying very little force.

Following this, attach the hand straps using the micro Velcro provided. Observe here that the patient still feels good and is not strangulated. Within the course of this fitting process, the BORT Generation Back Support is fully fitted to the anatomy of the body and interlocks.

If the lower part of the support ends with the hip-bend when sitting, the support is correctly positioned.

To take off the support, detach the Velcro on the pulley hand straps and then open the front fastener.

Indications:

Degenerative lumbar changes, mild to moderate lumboischialgia, slipped disc with mild deficits, spondylosis, spondylolisthesis, spinal canal stenosis, radicular irritation syndrome, condition following intervertebral disc surgery.

Contraindications:

Lymphatic drainage disorders including unclear soft tissue swelling, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body.

Composition:

84 % polyamide, 16 % spandex

Important note:

No reutilisation – this medical device is meant for the care of one patient only. Wear splint only when under strain, not during rest or sleep. Avoid hindering blood circulation. In case of numbness loosen splint and remove it if necessary. If complaints continue, please contact the doctor.

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