BORT Generation Hüftprotektor

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Generation

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



REF 215 900

Illustrationen
Figures


1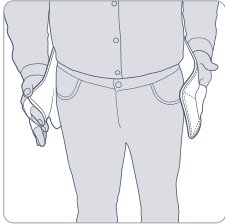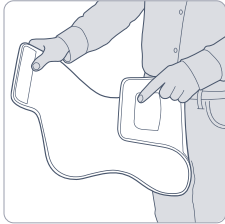

2

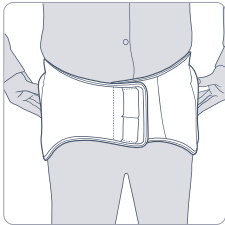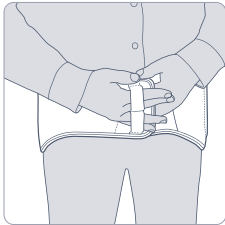3

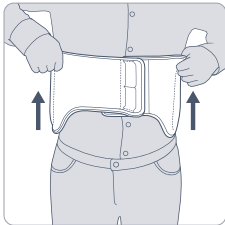Sprachen Languages

DE	Deutsch	Gebrauchsanweisung	04
EN	English	Instructions for use	08
FR	Français	Mode d'emploi	12
ES	Español	Instrucciones de uso	16
IT	Italiano	Istruzioni per l'uso	20
NL	Nederlands	Gebruiksaanwijzing	24
CS	Čeština	Návod k použití	28
ET	Eesti	Kasutusjuhend	32
PL	Polski	Instrukcja użytkowania	36
RO	Română	Instrucțiuni de utilizare	40

 PDF: ga.bort.com

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Many thanks for placing your trust in a medical device from BORT GmbH. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this medical device.

Intended purpose

This medical device is a hip protector for reducing the forces which influence the hip/femoral neck impact absorption in case of a fall.

Indications

Tendency to fall for various reasons, danger of falling.

Contraindications

Unclear soft tissue swellings, impaired sensation and circulation in the specific body region, skin diseases in the specific part of the body.

Application risks/Important notes

This medical device is a prescribable product. Discuss the use and duration with your treating physician. The expert staff from whom you have received the medical device will select the appropriate size and instruct you regarding its use.

- remove the medical device prior to radiological examinations
- if wearing at night is prescribed, do not close it too tightly
- in case of numbness, loosen the medical device and remove it if necessary
- in case of persistent complaints, consult the physician or specialist retailer
- use the medical device in accordance with therapeutic needs
- only use other products simultaneously after consultation with your physician
- do not make any changes to the product
- do not wear it on open wounds
- do not use in case of intolerance of one of the materials used
- no re-use. This medical aid is intended for treating one patient only

- whilst wearing the protector, please neither use any creams nor ointments on or around it as they can destroy the material

Fitting instructions

The BORT Generation hip protector can be worn above and below clothes.

- 1** Before putting it on, open the Velcro fastening at the front and guide the belt, if possible using the hand straps, from the rear around the pelvis with a slight tension.
- 2** Close the Velcro fastening and then check the correct positioning of the protector cushions in the side area of the trochanter major.
- 3** To guarantee as simple as possible handling in everyday life, the BORT Generation hip protector can be pulled upwards when closed and going to the toilet and then back to the correct position when ready.

Removal

To remove the hip protector, open the Velcro fastener. Close the Velcro fastener in order to avoid early wear to the Velcro surface.

Material composition

Polyamide (PA), elastane (EL)

The sewn in textile label provides the precise material composition.

Cleaning information

 Delicate wash 30 °C  Do not bleach  Do not dry in a tumble dryer
 Do not iron  Do not clean chemically

Remove the protector pads before washing. Close the Velcro fastening to avoid damage to other laundry items. Do not use fabric conditioner. Stretch

back into shape and dry in the fresh air. Hand-wash the protection pads with lukewarm water and mild detergents. When then re-inserting the pads, please observe that the concave cushion side must point towards the body.

Guarantee

The legal regulations of the country in which you acquired the product apply to the purchased product. Please contact your specialist retailer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been properly observed, the warranty may be impaired or cancelled. The warranty does not cover use of the product inappropriate for the indication, non-observance of application risks, instructions and unauthorised modifications to the product.

Useful life/Lifetime of the product

The lifetime of the medical device is determined by its natural wear and tear if treated appropriately and as recommended.

Duty of notification

If a serious worsening of the state of health occurs when using the medical device, you are obliged to notify the specialist dealer or us as the manufacturer and the Medicines & Healthcare products Regulatory Agency (MHRA).

You can find our contact information in these instructions for use. You can find the contact information for the national authority responsible for your country under the following link: www.bort.com/md-eu-contact.

Disposal

Upon the termination of use, the product must be disposed of in accordance with the corresponding local requirements.

Declaration of conformity

We confirm that this device conforms with the requirements of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL. You can find the current declaration of conformity under the following link: www.bort.com/conformity

Status: 05.2023

MD Medical device |  Single patient – multiple use

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