Generation Hüftprotektor

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

Hip protectors absorb or distribute the forces which influence the hip and therefore reduce the risk of a fracture of the femoral neck.

The BORT Generation Hip Protector consists of two coordinated components: the functional protector belt and the protector set made of special shock-absorbing foam. Combined, they provide a preventive measure and can considerably reduce the fracture risk in case of a fall.



The BORT Generation Hip Protector is characterised by its easy handling and high comfort to wear. The foam padding cushions huddle against the body with the three-dimensional design and the special properties of the material used and therefore guarantee an optimum fit.

Instructions for use:


The BORT Generation Hip Protector can be worn above and below clothes.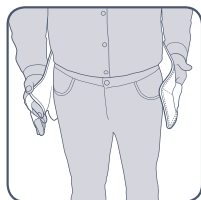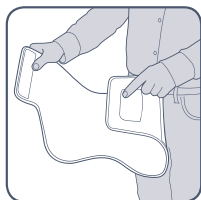

Before putting it on, open the Velcro fastening at the front and guide the belt, if possible using the hand straps, from the rear around the pelvis with a slight tension.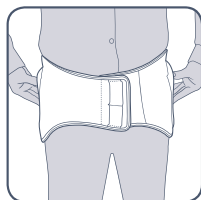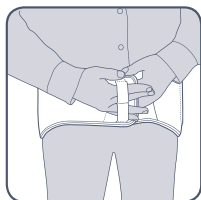

Close the Velcro fastening and then check the correct positioning of the protector cushions in the side area of the trochanter major.

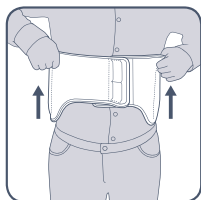To guarantee as simple as possible handling in everyday life, the BORT Generation Hip Protector can be pulled upwards when closed and going to the toilet and then back to the correct position when ready.

Indications:

Tendency to fall for various reasons, danger of falling.

Contraindications:

Lymphatic drainage disorders including unclear soft tissue swelling, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body.

Composition:

84 % polyamide, 16 % spandex

Important Instructions:

No reutilisation – this medical device is meant for the care of one patient only. Wear splint only when under strain, not during rest or sleep. Avoid hindering blood circulation. In case of numbness loosen splint and remove it if necessary. If complaints continue, please contact the doctor.

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Hand-wash the support in lukewarm water using a mild detergent. Do not use fabric softeners. Close the Velcro closure to avoid damaging other items being washed. Shape by hand and allow dry in the open air. Avoid heat. For washing, remove both protector pads upwards out of the side pockets. When then re-inserting the pads, please observe that the concave cushion side must point towards the body.

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