Generation Hallux-Valgus-Schiene

Hallux Valgus Splint | Éclisse hallux valgus | Férula Hallux Valgus |
Stecca per alluce valgo Hallux Valgus

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
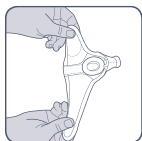
The BORT Generation Hallux Valgus Splint serves corrective or postoperative stabilisation of the large toe and the metatarsophalangeal joint. The support can be worn both during the daytime and at night. The padded relief ring should be positioned precisely at the same level as the metatarsophalangeal joint, thus providing pressure relief for the metatarsophalangeal joint. The laterally positioned corrective support enables individual correction of the metatarsophalangeal joint. Depending on the use and degree of severity of the metatarsophalangeal position, it is possible to insert an elastic spiral spring or a rigid aluminium splint in the side velour pocket intended for this purpose. The use of thin materials provides for superior comfort and it can therefore be easily worn in footwear. The wearing time can vary from patient to patient and is determined by the existing anatomical circumstances. Remove the brace at regular intervals and check the skin appearance for redness or pressure sores.

Instructions for use:

Open the BORT Generation Hallux Valgus Splint fasteners fully.

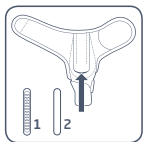The relief pad is attached to the support with Velcro and can be freely positioned as needed.

Select the stabilisation element and slide it fully into the velour pocket intended for this purpose.

Static and dynamic application possibilities, configuration-dependent: 1. Spiral spring (flexible), 2. Splint (rigid)

In the next step, pull the support over the large toe (hallux). Here, the relief pad should be precisely on the same level as the metatarsophalangeal joint.

Close the BORT Generation Hallux Valgus Splint over the instep of the foot with an even pull. Pay attention to pleasant wearing comfort.

Pull the laterally positioned corrective support towards the heel with a well-dosed pull, here a visible correction of the hallux valgus (large toe position discrepancy) should take place.

Indications:

Hallux valgus, after hallux surgery to ensure the success of treatment, post-operative capsule relief.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, intolerance of one of the materials used.

Composition:

86 % polyamide, 14 % spandex

Important note:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

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