
select® AchilloStabil® Plus Achillessehnenbandage

Achilles' Tendon Support | Chevillère | de soutien du talon d'Achille
Ortesis para el tendón de Aquiles | Fascia per il tendine di Achille

- ⊕ SOFTflex Plus-Funktionsgestrick
- ⊕ Kühlende **BREEZE** Komfort-Faser
- ⊕ Intermittierende Massage

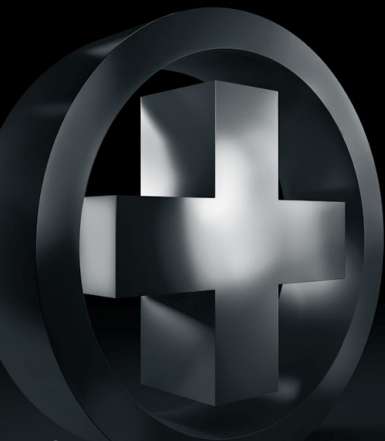select®

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Art.-No.
053 100

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The first choice for most demanding people.

select[®] - our highest-quality series of supports convinces for its functionality and the excellent product components. It offers unique characteristics, which satisfy even the most demanding patients.

select[®]

BORT. Benefit at your side.

EN Dear Customer,

Thank you for choosing a premium medical product manufactured by BORT GmbH. With the **BORT select® AchilloStabil® Plus Ankle Support**, your specialist medical provider has supplied you with a product that meets the highest medical, functional and quality standards.

Selected high-quality materials together with cooling luxury **BREEZE** fibre meet the highest standards and ensure patient-friendly use.

The BORT **select® AchilloStabil® Plus Ankle Support** relieve injured tissue structures. The support's compressive action supports the postural function of the ligaments thus relieving the joint. At the same time, the support helps prevent incorrect movement.

The viscoelastic pad in the Achilles tendon region promotes lymph drainage and blood circulation by its massage effect.

USING THE SLIP-ON AID / PULL-ON AID

Slip your foot into the pull-on aid. To easily put on a support, slide it over the slip-on aid onto your foot. Make sure the support is positioned correctly and that the Achilles tendon is well embedded by the pad.

Afterwards, simply pull the slip-on aid forwards out of the support.

INDICATIONS

Chronic or post-traumatic irritation or tenderness in the Achilles tendon area, Haglund syndrome heel irritation, achillodynia, irritation and pain of the Achilles tendon and surrounding areas, after surgery, for example in the case of Achilles tendon rupture, bursitis subachillea.

CONTRAINDICATIONS

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

COMPOSITION

66 % Polyamide

29 % Elastodies/Latex

5 % Spandex

IMPORTANT NOTE

No reutilisation – this medical device is meant for the care of one patient only.

The **BORT select® AchilloStabil® Plus Ankle Support** supports the healing process particularly during activities. If instructed to wear the support at night, you must take precautions to avoid impairment of blood circulation. If you experience numbness, please loosen the support immediately and, if necessary, remove it.

If you suffer from persistent complaints or if you experience worsening complaints, you should consult with your doctor.

Please do not apply any creams or ointments at any time while wearing the support as these can destroy the material.

The product contains latex and can trigger allergic reactions.

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BORT GmbH | Postfach 1330 | D-71367 Weinstadt | www.bort.com