select®

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# The first choice for most demanding people.

**select**<sup>®</sup> - our highest-quality series of supports convinces for its functionality and the excellent product components. It offers unique characteristics, which satisfy even the most demanding patients.

**select**<sup>®</sup>

**BORT. Benefit at your side.**

**GB** Dear Customer,

we are happy that you have selected a high-quality medical product from BORT GmbH. With the **BORT select® back support with a pad**, you have received a product from your medical specialist dealer which meets the highest medical, functional and qualitative requirements.

The **BORT select® back support with a pad** supports damaged tissue structures (bones, ligaments and cartilage).

The compression supports the retaining function of the ligaments at the same time, thus relieving the vertebrae.

The pad provides a massage effect which reduces the swelling tendency or contributes to the faster regression of existing swelling.

## **INSTRUCTIONS FOR USE**

Place the **BORT select® back support with a pad** around the waist and attach both closing sides using Velcro evenly one above the other with a slight pull of the hand straps.

Ensure that the support is placed centrally and the lower edge of the support ends in the inguinal region (forward bend). This is the only means to achieve the optimum effect from the **BORT select® back support**.

The pad is attached to the support with Velcro and can be freely positioned. Ensure that the wide side of the pad points towards the head.

## **INDICATIONS**

Pain in the lumbar spine, recurrent low back pain, osteochondrosis, spondylarthrosis, spondylolysis without spondylolisthesis, mild lumbar deformity (degenerative spondylolisthesis), dorsalgia.

## **CONTRAINDICATIONS**

Lymphatic drainage disorders including unclear soft tissue swelling, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body.

## COMPOSITION

55 % spandex, 45 % polyamide

## IMPORTANT NOTE

No re-use – this medical aid is intended for treating one patient.

The **BORT select® back support with a pad** supports the healing process especially during activity. If wearing at night is prescribed, negative influence on the circulatory system must be avoided. In case of numbness, loosen the support immediately and remove it if necessary.

In case of persistent or worsening complaints, please consult your doctor.

Whilst wearing the support, please neither use any creams nor ointments as they can destroy the material.

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BORT GmbH | Postfach 1330 | D-71367 Weinstadt | [www.bort.com](http://www.bort.com)