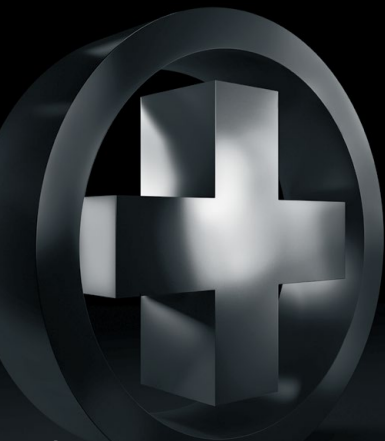
select® ManuZip® volar Handgelenkbandage

Wrist Support | Bandage pour poignet | Munequera
Fasciatura per articolazioni

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The first choice for most demanding people.

select[®] - our highest-quality series of supports convinces for its functionality and the excellent product components. It offers unique characteristics, which satisfy even the most demanding patients.

select[®]

BORT. Benefit at your side.

EN Dear Customer,

Thank you for choosing a premium medical product manufactured by BORT GmbH. With the **BORT select® ManuZip® volar Wrist Support**, your specialist medical provider has supplied you with a product that meets the highest medical, functional and quality standards.

The **BORT select® ManuZip® volar Wrist Support** supports the damaged tissue structures. The support's compressive action supports the postural function of the ligaments thus relieving the joint. At the same time, the support helps prevent incorrect movement.

On the move, the compressive knitting and the silicone pads combine to deliver a massage effect, which reduces the tendency for swelling, helps reduce existing swellings and thus accelerates the healing process.

An adjustable volar splint and the Stabilo® strap, whose pressure can be adjusted to suit the patient, offer safe stabilization of the wrist.

INSTRUCTIONS FOR USE

Bend aluminium splint to the desired form before putting it on.

The thumb opening can be opened by unpicking the thread facing the thumb.

The circular Stabilo® strap has two differently coloured Velcro surfaces. Always fasten the strap with the transparent area on the fastening point of the brace.

INDICATIONS

Chronic, post-traumatic or post-operative irritations in the wrist area, i.e. Sprains, Carpal arthrosis, Tenosynovitis.

CONTRAINDICATIONS

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

COMPOSITION

68 % Polyamide

25 % Elastodies

7 % Viscose

IMPORTANT NOTE

No reutilisation – this medical device is meant for the care of one patient only.

The **BORT select® ManuZip® volar Wrist Support** supports the healing process particularly during activities. If instructed to wear the support at night, you must take precautions to avoid impairment of blood circulation. If you experience numbness, please loosen the support immediately and, if necessary, remove it.

If you suffer from persistent complaints or if you experience worsening complaints, you should consult with your doctor.

Please do not apply any creams or ointments at any time while wearing the support as these can destroy the material.

The product contains latex and can trigger allergic reactions.

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BORT GmbH | Postfach 1330 | D-71367 Weinstadt | www.bort.com