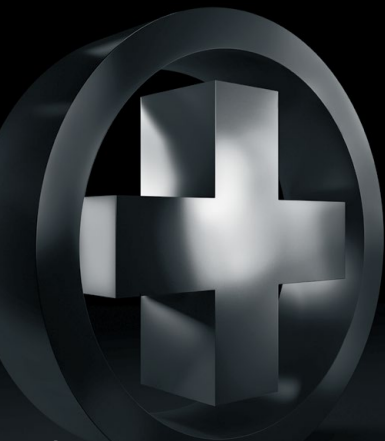
select® EpiPlus® Ellenbogenbandage

Elbow Support | Bandage pour coude | Codera | Fasciatura per gomiti

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The first choice for most demanding people.

select[®] - our highest-quality series of supports convinces for its functionality and the excellent product components. It offers unique characteristics, which satisfy even the most demanding patients.

select[®]

BORT. Benefit at your side.

EN Dear Customer,

Thank you for choosing a premium medical product manufactured by BORT GmbH. With the **BORT select® EpiPlus® Elbow Support**, your specialist medical provider has supplied you with a product that meets the highest medical, functional and quality standards.

Selected high-quality materials together with cooling luxury **BREEZE** fibre meet the highest standards and ensure patient-friendly use.

The **BORT select® EpiPlus® Elbow Support** supports the damaged tissue structures. The support's compressive action supports the postural function of the ligaments thus relieving the joint. At the same time, the support helps prevent incorrect movement.

On the move, the compressive knitting and the silicone pads combine to deliver a massage effect, which reduces the tendency for swelling, helps reduce existing swellings and thus accelerates the healing process. Pressure applied at specific points has a pain-relieving effect and relieves strain on the ligament attachments.

INSTRUCTIONS FOR USE

Fit the **BORT select**® **EpiPlus**® **Elbow Support** in such a way that the comfort zone is positioned in the crook of the arm. Make sure that you can touch the outer and inner articular process through the gaps in the silicone pad.

The circular **Stabilo**® strap allows further adjustment of the pressure on the silicone pads.

INDICATIONS

Chronic post-traumatic or post-operative irritations of soft tissues in the area of the elbow joint, Epicondylitis (acute or chronic), tendomyopathy.

CONTRAINDICATIONS

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

COMPOSITION

66 % Polyamide

29 % Elastodies/Latex

5 % Spandex

IMPORTANT NOTE

No reutilisation – this medical device is meant for the care of one patient only.

The **BORT select**® **EpiPlus**® **Elbow Support** supports the healing process particularly during activities. If instructed to wear the support at night, you must take precautions to avoid impairment of blood circulation.

If you experience numbness, please loosen the support immediately and, if necessary, remove it.

If you suffer from persistent complaints or if you experience worsening complaints, you should consult with your doctor.

Please do not apply any creams or ointments at any time while wearing the support as these can destroy the material.

The product contains latex and can trigger allergic reactions.

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BORT GmbH | Postfach 1330 | D-71367 Weinstadt | www.bort.com