EpiContur® Sport

Epicondylitis-Soft-Spange

Epicondylitis soft clasp | Bracelet épicondylite soft |
Codera para epicondilitis | Presa morbida per epicondilitite

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The **epicondylitis soft clasp EpiContur® Sport from BORT** is an effective aid for relief in case of epicondylitis, the so-called tennis or golfer's arm. The clasp is positioned approximately 5 cm below the elbow joint on the lower arm and relieves the structures affected.

Instructions for use:

- You can position the pad freely with the Velcro pad. Markings are printed on the support, showing the correct positioning. The narrower area of the pad should be placed in the direction of the elbow joint. For this purpose, guide the Velcro tape through the plastic eyelet.
- Then close the clasp so that it fits comfortably on the lower arm, but does not cause any congestion.
- The pad should only exert pressure on the muscle belly when you make a fist or a grasping movement.
- When the hand is relaxed, the BORT EpiContur® Sport clasp should be positioned without perceptible pad pressure.

Indications:

Epicondylitis, chronic or recurring epicondylopathy, tendinopathy, strain as a consequence of work and sports, so-called tennis/golfer's arm.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders including unclear soft tissue swelling distal to the positioned aid, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body.

Composition:

54 % polyamide, 40 % polyurethane, 6 % polyester

Important Note:

No re-use – this medical aid is intended for treating one patient.

If the clasp is to be worn at night, observe that the circulation is not impeded. In case of feelings of numbness, loosen the clasp and remove it if necessary. In case of persistent complaints, you should consult a doctor.

Neither use any creams nor ointments when wearing the support as they could damage the material.

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After taking out pad, wash by hand in hand warm water with lightduty
detergent. Fix the Velcro closure to avoid damage to other laundry. Pull
into shape and air-dry.

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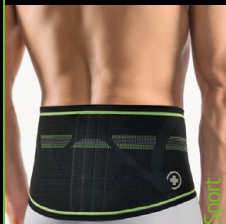
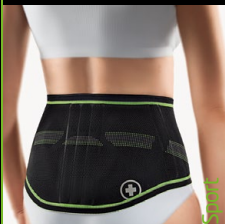
EN The **BORT EpiContur® Sport** for sports enthusiasts relieves the end points of the forearm muscles and elbow tendons. The individually adjustable brace pressure can ensure the required pain relief in case of burdens, thus making it very comfortable to wear.

BORT Quality products promote your healing process: in the fields of orthopedics, surgery, sports medicine and rehabilitation.

BORT GmbH | Postfach 1330 | D-71367 Weinstadt | www.bort.com

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