AchilloStabil® Plus Sport

Aktiv-Achillessehnenbandage

Active Achilles tendon support | Chevillière du tendon d'Achille active |
Tobillera Aktiv para el tendón de Aquiles | Cavigliera attiva per il tendine di Achille

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The **AchilloStabil® Plus Sport support from BORT** for Achilles tendon compression supports damaged tissue structures (joints, ligaments, capsules and cartilage), in doing so strengthens the retaining function of the ligaments, thus relieving the joint. At the same time, this support helps to avoid incorrect movement to a large extent. The extended structured silicone pad results in a massage effect which reduces the swelling tendency or contributes to the faster regression of existing swelling.

The snag proof material edge at the calf combined with the SOFTflex 3D knitted fabric ensures a really soft and smooth fit.

COOLMAX® fibre wicks moisture away from the body, ensuring dry and pleasant wearing comfort.

Instructions for use:

- Grasp the upper edge of the support and pull it over your foot.
- Ensure that the profile insert fully covers the Achilles tendon.
- Correct the positioning of the support as applicable.

Indications:

Chronic or post-traumatic irritation or tenderness in the Achilles tendon area, for example in the case of Haglund syndrome heel irritation, achillodynia, condition after surgery, for example in the case of Achilles tendon rupture or bursitis subachillea.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders including unclear soft tissue swelling distal to the positioned aid, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body.

Composition:

40 % polyamide (**COOLMAX®**), 40 % polyamide, 19 % elastane (LYCRA®), 1 % silicone

Important Note:

No re-use – this medical aid is intended for treating one patient.

If the support is to be worn at night, observe that the circulation is not impeded. In case of feelings of numbness, loosen the support and remove it if necessary. In case of persistent complaints, you should consult a doctor.

Neither use any creams nor ointments when wearing the support as they could damage the material.

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EN The **BORT AchilloStabil® Plus Sport Achilles tendon support** relieves the painful and inflammatory Achilles tendon without restricting movement. The viscoelastic silicone pad massages and stimulates the muscle-tendon passages, thus actively supporting the healing process. The **SOFTflex 3D knitted fabric** and moisture regulating properties of the high-tech **COOLMAX®** fibre ensure that it is very comfortable to wear and for ideal properties for sports activities.

BORT Quality products promote your healing process: in the fields of orthopedics, surgery, sports medicine and rehabilitation.

BORT GmbH | Postfach 1330 | D-71367 Weinstadt | www.bort.com

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