TaloStabil® Sport Sprunggelenkbandage

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The **ankle joint support Talostabil® Sport from BORT** supports damaged tissue structures (joints, ligaments, capsules and cartilage). Here, the compression strengthens the retaining function of the ligaments, thus relieving the joint. At the same time, this support helps to avoid incorrect movement. When on the move, the compressed SOFTflex 3D knitted fabric ensures greater elasticity and stability and is very comfortable to wear, whilst the two silicone pads cause a massage effect which reduces the swelling tendency or contributes to faster regression of existing swelling.

COOLMAX® fibre wicks moisture away from the body, ensuring dry and pleasant wearing comfort.

Instructions for use:

Grasp the upper edge of the support and pull it over your foot.

Indications:

Chronic post-traumatic or post-operative soft tissue irritations in the ankle area, distortion, ligament weakness, after removal of plaster cast, ligament injuries, upper ankle joint distortion, arthrosis.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders including unclear soft tissue swelling distal to the positioned aid, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body.

Composition:

40 % polyamide (**COOLMAX®**), 40 % polyamide, 19 % elastane (LYCRA®), 1 % silicone

Important Note:

No re-use – this medical aid is intended for treating one patient.

If the support is to be worn at night, observe that the circulation is not impeded. In case of feelings of numbness, loosen the support and remove it if necessary. In case of persistent complaints, you should consult a doctor.

Neither use any creams nor ointments when wearing the support as they could damage the material.

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EN The **BORT TaloStabil® Sport active support** relieves the weak or diseased ankle joint. Two viscoelastic pads stimulate circulation and help to reduce oedemas. The SOFTflex 3D knitted fabric and moisture regulating properties of the high-tech **COOLMAX®** fibre ensure that it is very comfortable to wear and for ideal properties for use during sports activities.

BORT Quality products promote your healing process: in the fields of orthopedics, surgery, sports medicine and rehabilitation.

BORT GmbH | Postfach 1330 | D-71367 Weinstadt | www.bort.com

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Stand: Oktober 2018

D054600SP/D-10/18