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The **back support StabiloBasic Lady Sport from BORT** serves to provide muscular support for the lumbar spine. The compression of the active support strengthens the retaining function of the ligaments, thus relieving the vertebrae. The fit and height of the support are adapted to the female anatomy and stabilised using four supporting splints. The viscoelastic sacral pad can be freely positioned, causing a massage effect which reduces the swelling tendency or contributes to the faster regression of existing swelling.

COOLMAX® fibre wicks moisture away from the body, ensuring dry and pleasant wearing comfort.

Instructions for use:

- Place the support around the waist and attach both closing sides using Velcro evenly one above the other with a slight pull of the hand straps.
- Ensure that the support is placed centrally and the lower edge of the support ends in the inguinal region (forward bend). This is the only means to achieve the optimum effect from the BORT StabiloBasic Lady Sport.
- The pad is attached to the support with Velcro and can be freely positioned. However, you must ensure that the wide side of the pad points towards the head.

Indications:

Pain in the lumbar spine, recurrent low back pain, osteochondrosis, spondylarthrosis, spondylolysis without spondylolisthesis, mild lumbar deformity (degenerative spondylolisthesis), ischialgia, dorsalgia.

Contraindications:

Lymphatic drainage disorders including unclear soft tissue swelling, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body.

Composition:

40 % polyamide (**COOLMAX®**), 40 % polyamide, 19 % elastane (LYCRA®), 1 % silicone

Important Note:

No re-use – this medical aid is intended for treating one patient.

If the support is to be worn at night, observe that the circulation is not impeded. In case of feelings of numbness, loosen the support and remove it if necessary. In case of persistent complaints, you should consult a doctor.

Neither use any creams nor ointments when wearing the support as they could damage the material.

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EN The **BORT StabiloBasic Lady Sport back support** stabilises the lumbar spine. A sacral pad which can be freely positioned gently massages affected areas in the sacral bone region. This supports the release of tension and can alleviate pain. The moisture regulating properties of the high-tech **COOLMAX®** fibre ensures that it is very comfortable to wear and for ideal properties for use during sports activities.

BORT Quality products promote your healing process: in the fields of orthopedics, surgery, sports medicine and rehabilitation.

BORT GmbH | Postfach 1330 | D-71367 Weinstadt | www.bort.com

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