Patella-Sehnenband Sport

Patella Tendon Support | Bande pour tendon rotulien |
Soporte para tendón rotuliano | Fascia per tendine rotuleo

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The **Patella Tendon Support Sport from BORT** exerts pressure on the tendon connected to the kneecap (patella). This helps the patella to glide in the correct path during knee movement. Simultaneously, the pressure results in increased pressure of the musculature responsible for guiding the kneecap. You can adjust the pad pressure individually.

The Patella Tendon Support Sport with its modern design has no guide loop and is particularly suitable for high activity levels and also contact sport types. The soft, elastic strap material ensures high wearing comfort.

Instructions for use:

- Open the patella tendon support and place it directly beneath the kneecap.
- Close the Velcro tape on the patella tendon support and ensure that sufficient pressure is exerted on the kneecap tendon whilst avoiding causing congestions in the lower leg area.

Indications:

Patellar chondropathy, femoropatellar arthrosis, patellar tendonitis.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders including unclear soft tissue swelling distal to the positioned aid, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body.

Composition:

84% polyamide, 16% elastane

Important note:

No re-use – this medical aid is intended for treating one patient.

If the patella tendon support is to be worn at night, observe that the circulation is not impeded. In case of feelings of numbness, loosen the patella tendon support and remove it if necessary. In case of persistent complaints, you should consult a doctor.

Neither use any creams nor ointments when wearing the support as they could damage the material.

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EN The **BORT Patella tendon support Sport** has a direct effect on the patella tendon via its inserted, noded silicone pad. This enables considerable alleviation of instability and pain in the regions of the kneecap and improvement in coordination and muscle building during sports activity within the course of the rehabilitation phase following an injury. Pad pressure can be individually metered.

BORT Quality products promote your healing process: in the fields of orthopedics, surgery, sports medicine and rehabilitation.

BORT GmbH | Postfach 1330 | D-71367 Weinstadt | www.bort.com

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