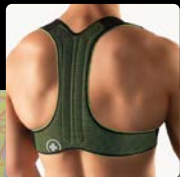
# BORT Geradehalter Sport

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
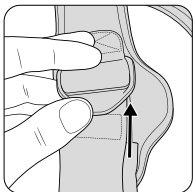
Sport

Illustrationen  
Figures



1



2

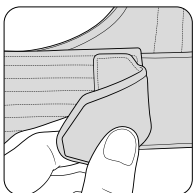3

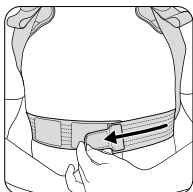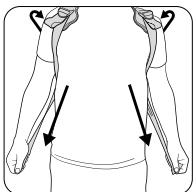4
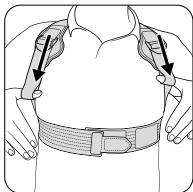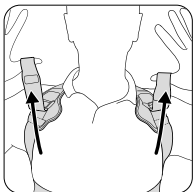
## Sprachen Languages

<b>DE</b>	Deutsch	Gebrauchsanweisung .....	04
<b>EN</b>	English	Instructions for use .....	08
<b>FR</b>	Français	Mode d'emploi .....	12
<b>ES</b>	Español	Instrucciones de uso .....	16
<b>IT</b>	Italiano	Istruzioni per l'uso .....	20
<b>NL</b>	Nederlands	Gebruiksaanwijzing .....	24
<b>CS</b>	Čeština	Návod k použití .....	28
<b>ET</b>	Eesti	Kasutusjuhend .....	32
<b>PL</b>	Polski	Instrukcja użytkowania .....	36
<b>RO</b>	Română	Instrucțiunile de utilizare .....	40

PDF: [ga.bort.com](http://ga.bort.com)

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Many thanks for placing your trust in a medical device from BORT GmbH. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this medical device.

### **Intended purpose**

This device is a brace for straightening and relieving the thoracic spine.

### **Indications**

Straightening and relief of the thoracic spine, e.g. misalignment of the thoracic spine, dorsalgia (pain in the thoracic spine region)

### **Contraindications**

Thrombosis risk, extreme varicosis, peripheral arterial occlusive disease (PAOD), lymph drainage disorders, also unclear soft tissue swellings distal to the aid positioned, sensory and circulatory disorders of the region of the body treated, skin diseases in the part of the body treated, latex allergy.

### **Application risks/Important notes** ⚠

This medical device is a prescribable product. Discuss the use and duration with your treating physician. The expert staff from whom you have received the medical device will select the appropriate size and instruct you regarding its use.

- remove the medical device prior to radiological examinations
- remove it during longer periods of rest, e.g. when sleeping
- in case of numbness, loosen the medical device and remove it if necessary
- in case of persistent complaints, consult the physician or specialist retailer
- use the medical device in accordance with therapeutic needs
- only use other products simultaneously after consultation with your physician
- do not make any changes to the product
- do not wear it on open wounds
- do not use in case of intolerance of one of the materials used
- no re-use. This medical aid is intended for treating one patient only
- whilst wearing the brace, please neither use any creams nor ointments on or around it as they can destroy the material.

## Fitting instructions

- 1** Loosen the Velcro which fixes the thumb loop on the restraint and pull the loop upwards to the rucksack buckle.
- 2** Open the chest strap using the Velcro fastener.
- 3** Guide the brace around the shoulders like a rucksack. Pull the brace over the chest straps and close the Velcro below the chest.
- 4** Then, actively straighten your thoracic spine, guide your thumbs through the thumb loops and release the Velcro by guiding the loops upwards. Following this, pull the restraints downwards tightly and fix the strap on the course of the strap using the Velcro.

## Removal

Remove the thumb loop from the Velcro and guide it to the rucksack buckle, thus loosening the restraint guidance. Open the chest strap and slip the corset off over the back as for a rucksack.

## Material composition

Polyamide (PA), polyester (PES), elastane (EL)

## Cleaning information

 Delicate wash 30 °C  Do not bleach  Do not dry in a tumble dryer  
 Do not iron  Do not clean chemically

**Information:** Before cleaning, pay attention to remove the splints in the back plate in order to avoid damages to the brace and your washing machine. Close the Velcro fastening to avoid damage to other laundry items. Do not use fabric conditioner. Stretch back into shape and dry in the fresh air.

**Guarantee**

The legal regulations of the country in which you acquired the product apply to the purchased product. Please contact your specialist retailer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been properly observed, the warranty may be impaired or cancelled. The warranty does not cover use of the product inappropriate for the indication, non-observance of application risks, instructions and unauthorised modifications to the product.

**Useful life/Lifetime of the product**

The lifetime of the medical device is determined by its natural wear and tear if treated appropriately and as recommended.

**Duty of notification**

If a serious worsening of the state of health occurs when using the medical device, you are obliged to notify the specialist dealer or us as the manufacturer and the Medicines & Healthcare products Regulatory Agency (MHRA).

You can find our contact information in these instructions for use. You can find the contact information for the national authority responsible for your country under the following link: [www.bort.com/md-eu-contact](http://www.bort.com/md-eu-contact).

**Disposal**

Upon the termination of use, the product must be disposed of in accordance with the corresponding local requirements.

### Declaration of conformity

We confirm that this device conforms with the requirements of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL.

You can find the current declaration of conformity under the following link:  
[www.bort.com/conformity](http://www.bort.com/conformity)

Status: 05.2023

**MD** Medical device |  Single patient – multiple use

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