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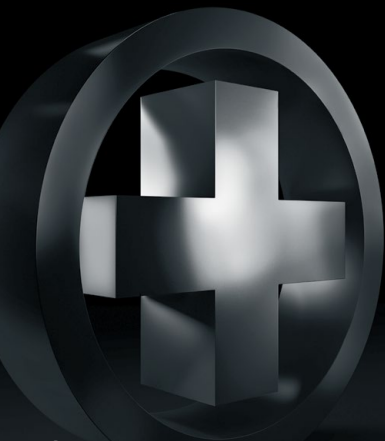
The first choice for most demanding people.

select[®] - our highest-quality series of supports convinces for its functionality and the excellent product components. It offers unique characteristics, which satisfy even the most demanding patients.

select[®]

BORT. Benefit at your side.

EN Dear Customer,

Thank you for choosing a premium medical product manufactured by BORT GmbH. With the **BORT select® Stabilo® Back Support** with Pad, your specialist medical provider has supplied you with a product that meets the highest medical, functional and quality standards.

The **BORT select® Stabilo® Back Support** with Pad support the damaged tissue structures. The supports compressive action supports the postural function of the ligaments thus relieving the spine.

The integrated cross-strap provides additional stabilisation and upright support of the lumbar spinal area. This improves the relief to the lumbar spinal area and can be fully and individually adjusted.

The pad provides an additional massage effect, which reduces the tendency for swelling or helps accelerate the reduction of existing swellings.

INSTRUCTIONS FOR USE

Place the **BORT select® Stabilo® Back Support** with Pad around your waist and, using the hand straps, close the front closure.

Make sure it fits comfortably. To provide further relief to the lumbar region, simultaneously and evenly tighten the side straps.

To avoid undesirable rotation around the spine, make sure that you pull both side straps simultaneously.

The pad is velcroed to the support and can be positioned freely. Ensure that the wide side of the pad faces upward.

INDICATIONS

Treatment of increased pain in the lumbar spine area, severe low back pain, recurrent low back pain, osteochondrosis, spondylarthrosis, lumbosciatica, mild radicular syndrome, spondylolysis without spondylolisthesis, mild to moderate lumbar deformity (degenerative spondylolisthesis), after intervertebral disc surgery.

CONTRAINDICATIONS

Lymphatic drainage disorders including unclear soft tissue swelling, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body, latex allergy.

COMPOSITION

68 % Elastodics

32 % Polyamide

IMPORTANT NOTE

No reutilisation – this medical device is meant for the care of one patient only.

The **BORT select® Stabilo® Back Support** with Pad supports the healing process particularly during activities. If instructed to wear the support at night, you must take precautions to avoid impairment of blood circulation. If you experience numbness, please loosen the support immediately and, if necessary, remove it.

If you suffer from persistent complaints or if you experience worsening complaints, you should consult with your doctor.

Please do not apply any creams or ointments at any time while wearing the support as these can destroy the material.

The product contains latex and can trigger allergic reactions.

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BORT GmbH | Postfach 1330 | D-71367 Weinstadt | www.bort.com