StabiloGen® Sport

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Knee support with a silicone pad | Genouillère avec pelote en silicone |
Rodillera con almohadilla de silicona | Ginocchiera con pelotta in silicone

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The **knee support StabiloGen® Sport from BORT** supports damaged tissue structures (joints, ligaments, capsules and cartilage). Here, the compression from the support strengthens the retaining function of the ligaments, thus relieving the joint. At the same time, this support helps to avoid incorrect movement.

The pad results in a massage effect which reduces the swelling tendency or contributes to the faster regression of existing swelling. The kneecap is guided by the pad in its normal movement sequence.

Thanks to the Plus size system with a silicone adhesive border, a good fit is guaranteed. One spiral spring stay one each side respectively ensures a good fit and the SOFTflex 3D knitted fabric provides more elasticity and stability with a highly-elastic, soft flexion region in the hollow of the knee. **COOLMAX®** fibre wicks moisture away from the body, ensuring dry and pleasant wearing comfort.

Instructions for use:

- Grasp the upper edge of the support and pull it over your foot up to your knee with the pad pointed forwards.
- The pad should be placed comfortably over the kneecap.

Indications:

Chronic post-traumatic or post-operative irritations of knee joint soft tissues, recurrent articular effusion, e.g. in the case of gonarthrosis, arthritis, ligament instability, meniscopathy.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders including unclear soft tissue swelling distal to the positioned aid, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body.

Material composition:

40 % polyamide (**COOLMAX®**), 40 % polyamide, 19 % elastane (LYCRA®), 1 % silicone

Important note:

No re-use – this medical aid is intended for treating one patient.

If the support is to be worn at night, observe that the circulation is not impeded. In case of feelings of numbness, loosen the support and remove it if necessary. In case of persistent complaints, you should consult a doctor.

Neither use any creams nor ointments when wearing the support as they could damage the material.

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Hand-wash the support in lukewarm water using a mild detergent. Do not use fabric softeners. Pull into shape and air-dry. Avoid heat.

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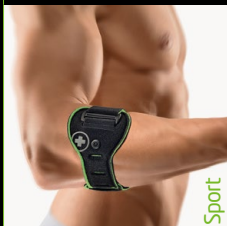
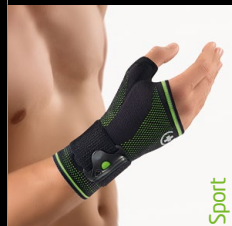
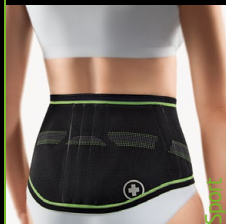
EN The **BORT StabiloGen® Sport** combines the stabilising function of a knee support with the moisture regulating properties of the high-tech **COOLMAX®** fibre. The **SOFTflex 3D** knitted fabric with a highly elastic flexion region in the hollow of the knee, the knitted silicone adhesive zone and the **Plus size system** make it comfortable to wear, providing an ideal fit for sports activities.

BORT Quality products promote your healing process: in the fields of orthopedics, surgery, sports medicine and rehabilitation.

BORT GmbH | Postfach 1330 | D-71367 Weinstadt | www.bort.com

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