IT

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EN

The BORT OsoTract® Upper Arm/Shoulder Brace is a special upper arm brace with 4 weights, 250 g each. The weights are individually inserted in pockets fitted with zip fasteners. This allows individual weight positioning.

Instructions for use:

Important: Support must be removed before radiological examinations (e.g. MRI, X-ray, CT)!

Remove individual weights out of pocket according to instructions To fit, open the Velcro straps and position the brace that the zip fastener points upwards towards the shoulder. Tighten Velcro closures, however avoid congestions.

Indications:

Impingement syndrome, humeroscapular periarthropathy (HSP).

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

Support: 61 % polyamide, 28 % polyurethane, 11 % microfibre polyester
Weights: Filling: 100 % steel, pocket: 80 % PA, 20 % spandex

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

Wear brace only when under strain, not during rest or sleep. Avoid hindering blood circulation. In case of numbness loosen brace and remove it if necessary. If complaints continue, please contact the doctor.

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