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EN

Treatment with the BORT Clavicula Support in the form of a backpack support provides secure fixation in conservative clavicular fracture therapy.

Instruction for use:

The BORT Clavicula Support should be fitted by a healthcare professional.

Place the support between the shoulder blades and draw the two padded straps over the shoulders.

Make sure the patient is in an upright body posture and draw the padded straps under the armpits to the back.

Take the tips of the straps through the guide loops and close the hook and loop fasteners.

Draw the vertical straps downwards and tension the support. Check the course and tension of the padded straps and tighten if necessary.

Indications:

Treatment of clavicle fractures.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

Composition:

50 % polyamide, 35 % PUR foam, 10 % natural rubber, 5 % cotton

Important Note:

No reutilization – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove if necessary. If complaints continue, please contact the doctor.

If pressure points, abrasions or similar occur, have your orthopaedic technician or doctor check the fit of the support immediately.

The product contains latex and can trigger allergic reactions.

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