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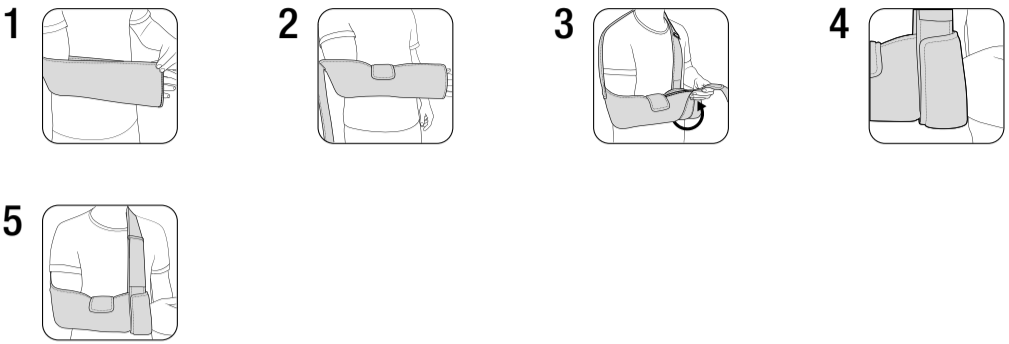
**REF** 121 989 CH

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D121989CHI2020-10I001 ML

### DE OmoSat ECO

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### EN OmoSat ECO

Many thanks for placing your trust in a medical device from BORT GmbH. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this medical device.

**Intended purpose**  
Shoulder-arm pouch for safe positioning of the arm in the adduction position.

**Indications**  
Sprain, contusion, injury to the rotators, luxation, subcapital humerus fracture, humerus head and humerus shaft fracture, after surgery.

**Contraindications**  
Thrombosis risk, extreme varicosis, peripheral arterial occlusive disease (PAOD), lymph drainage disorders, also unclear soft tissue swellings distal to the aid positioned, sensory and circulatory disorders of the region of the body treated, skin diseases in the part of the body treated.

**Application risks / Important notes**   
This medical device is a prescribable product. Discuss the use and duration with your treating physician. The expert staff from whom you have received the medical device will select the appropriate size and instruct you regarding its use.  
– remove the medical device prior to radiological examinations  
– remove it during longer periods of rest, e.g. when sleeping  
– in case of numbness, loosen the medical device and remove it if necessary  
– in case of persistent complaints, consult the physician or specialist retailer  
– use the medical device in accordance with therapeutic needs  
– only use other products simultaneously after consultation with your physician  
– do not make any changes to the product  
– do not wear it on open wounds  
– do not use in case of intolerance of one of the materials used  
– no re-use – this medical aid is intended for treating one patient  
– whilst wearing the support, please neither use any creams nor ointments on or around it as they can destroy the material

- Fitting instructions**
- Place the forearm in the arm pouch and position your elbow in the semi-circular stitched down area.
  - The arm pouch is closed at the height of the central lower arm region using the short forearm strap included. This strap fixes the forearm in the pouch.
  - The shoulder strap runs from the elbow area over the opposite side of the shoulder height and is then guided around the arm pouch at wrist level. The strap system Velcro fasteners enable individual setting of the desired resting position.
  - Optionally, the area of the arm pouch facing the hand can be turned up.

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**5** Check the correct fit of the support.

**Removal**  
To remove the OmoSat ECO shoulder joint support, open the Velcro fasteners in the reverse order and then remove the support.

**Material composition**  
Cotton (CO), polyamide (PA), polyester (PES)  
The sewn in textile label provides the precise material composition.

**Cleaning information**  
 Delicate wash 30° C Do not clean chemically Do not bleach Do not dry in a tumble dryer Do not iron

Close the Velcro fasteners in order to avoid damaging other items of clothing. Do not use fabric conditioner.

**Guarantee**  
The legal regulations of the country in which you acquired the product apply to the purchased product. Please contact your specialist retailer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been properly observed, the warranty may be impaired or cancelled. The warranty does not cover use of the product inappropriate for the indication, non-observance of application risks, instructions and unauthorised modifications to the product.

**Useful life/Lifetime of the product**  
The lifetime of the medical device is determined by its natural wear and tear if treated appropriately and as recommended.

**Duty of notification**  
If a serious worsening of the state of health occurs when using the medical device, you are obliged to notify the specialist dealer or us as the manufacturer and the Medicines & Healthcare products Regulatory Agency (MHRA). You can find our contact information in these instructions for use. You can find the contact information for the appointed authority for your country under the following link: [www.bort.com/md-eu-contact](http://www.bort.com/md-eu-contact).

**Disposal**  
Upon the termination of use, the product must be disposed of in accordance with the corresponding local requirements.

**Declaration of conformity**  
We confirm that this device conforms with the requirements of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL. You can find the current declaration of conformity under the following link: [www.bort.com/conformity](http://www.bort.com/conformity)

Status: 10.2020

Medical device | Single patient – multiple use

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