## Therapieketnet Standard

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**REF** 951 500/951 600

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## Therapieketnet Standard

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**REF** 951 500/951 600

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Stand: 10.2024

### EN BORT therapeutic putty standard

Many thanks for placing your trust in a medical device from BORT GmbH. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this medical device.

### Intended purpose

This medical device is a latex-free therapeutic putty with which specific finger and hand exercises achieve improved movement and function of the neuromuscular control loop.

### Indications

Restoration of freedom of movement in the hand.

### Contraindications

Skin diseases in the palm area.

**Application risks/Important notes** ⚠  
This medical device is a prescribable product (REF 951 500). Discuss the use and duration with your treating physician. The expert staff from whom you have received the medical device will select the appropriate resistance and instruct you regarding its use.

– use the medical device in accordance with therapeutic needs  
– only use other products simultaneously after consultation with your physician  
– do not make any changes to the product  
– do not use in case of open wounds in the palm area  
– do not use in case of intolerance of one of the materials used

– no re-use. This medical aid is intended for treating one patient only  
– Attention, unfit for human consumption  
– If the putty enters the eye, cleanse the eye with water and consult a physician if necessary  
– Not suitable for children under 3 years of age  
– Do not apply to clothing or further textiles (carpets) – in case of redness, pain or swellings, stop the exercise and consult a physician if necessary  
– Use with sweaty, cleaned or freshly disinfected hands may cause staining of the product.

### Instructions for use

Please use the therapeutic exercises in the enclosed exercise instructions. The required resistance of the putty depends on the degree of strength and is determined by the treating physician or therapist.

### Storage

Please store the therapeutic putty in the packaging provided at room temperature in order to avoid impurities.

### Material composition

Silicone polymer, Boracic acid, Mineral oil, Silicate, Glycerine, Pigments

### Cleaning information

The therapeutic putty cannot be cleaned. Please wash your hands thoroughly before use.

### Guarantee

The legal regulations of the country in which you acquired the product apply to the purchased product. Please contact your specialist retailer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been properly observed, the warranty may be impaired or cancelled. The warranty does not cover use of the product inappropriate for the indication, non-observance of application risks, instructions and unauthorised modifications to the product.

### Useful life/Lifetime of the product

The lifetime of the medical device is determined by its natural wear and tear if treated appropriately and as recommended.

### Duty of notification

If a serious worsening of the state of health occurs when using the medical device, you are obliged to notify the specialist dealer or us as the manufacturer and the Medicines & Healthcare products Regulatory Agency (MHRA). You can find our contact information in these instructions for use. You can find the contact information for the national authority responsible for your country under the following link: [www.bort.com/md-eu-contact](http://www.bort.com/md-eu-contact).

### Disposal

Upon the termination of use, the product must be disposed of in accordance with the corresponding local requirements.

### Declaration of conformity

We confirm that this device conforms with the requirements of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL. You can find the current declaration of conformity under the following link: [www.bort.com/conformity](http://www.bort.com/conformity)

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